



**Advanced
Wound Care
Dressings for
Progressive
Patient Outcomes**



Blue Silicone Tape is a medical grade silicone tape that is latex free with a snap-on protective cap available in different sizes. It's primary focus is exclusive to patients where the skin conditions are sensitive, fragile, or have a need to replace the dressing in the same location on the skin.

Blue Silicone Tape

The Best Surgery Scar and Wound Care

REF	UNIT OF MEASURE	CONTENTS	SOLD BY	SIZE
BST0515	Box (BX)	6 Rolls	1 Box of 6	5cm x 1.5m
BST0505	Box (BX)	6 Rolls	1 Box of 6	5cm x 5m
BST2505	Box (BX)	12 Rolls	1 Box of 12	2.5cm x 5m
BST2515	Box (BX)	12 Rolls	1 Box of 12	2.5cm x 1.5m

Product Benefits:

Gentle on Skin

Painless to remove

Waterproof

Strong adhesive without leaving a residual on skin

Does not stick to hair

Hypoallergenic

Pre-cut design for easy tear without instruments (scissors)

How to Apply:

+ Prepare the Skin:

Clean and dry skin to remove lotion, soil, and/or residue

+ Apply the Tape:

Apply tape to skin without stretching or tension

Apply firm pressure to activate adhesive and gain full contact with skin

+ Additional Tips:

Avoid touching adhesive surface during application to retain adhesion

Silicone Border Sacrum Wound Dressing

All-in-One Silicone Bordered Foam Dressing

The Silicone Border Sacrum is an all-in-one foam dressing that is designed especially for pressure ulcers and other acute and chronic wounds of the sacral area. It effectively absorbs and retains exudate and helps in maintaining a moist wound environment.



REF	UNIT OF MEASURE	CONTENTS	SOLD BY	SIZE
S71077	Box (BX)	5 Each	1 Box of 5	17.5 × 17.5cm
S71099	Box (BX)	5 Each	1 Box of 5	22.5 × 22.5cm

Product Features:

- Non-irritating adhesive for the healing of hard areas
- Anatomically shape to fit the lower back area, providing protection and comfort for extended periods
- Can absorb light to heavy levels of exudate
- Does not cause pain upon removal wounds

How to Apply:

- + Clean and dry skin and wound and surrounding area thoroughly
- + Remove the dressing from package and fold it in half
- + While holding side tabs together, remove lining
- + Position dressing over the wound and gently press to the skin
- + Gently remove dressings delivery system paper
- + Press the dressing to conform to the covered area



MedSupply Hydrocolloid are semiocclusive dressings that contains gel-forming components such as sodium carboxymethyl cellulose and gelatin. This inner layer promotes wound healing by absorbing exudate. This feature provides a moist healing environment, which allows for faster healing and reduced scar formation.

Hydrocolloid dressings are advanced wound care products that are waterproof and self-adhering making them easy to use and effective at maximizing healing.

Hydrocolloid Regular & Hydrocolloid Extra Thin Wound Dressing

REF	UNIT OF MEASURE	CONTENTS	SOLD BY	SIZE
G1308	Box (BX)	10 Each	1 Box of 10	10 × 10cm
G1315	Box (BX)	10 Each	1 Box of 10	15 × 15cm
G1317	Box (BX)	10 Each	1 Box of 10	15 × 20cm
G1318	Box (BX)	10 Each	1 Box of 10	20 × 20cm
G1408	Box (BX)	10 Each	1 Box of 10	10 × 10cm
G1415	Box (BX)	10 Each	1 Box of 10	15 × 15cm

Product Features:

Bacterial Barrier

Self-Adherent

Mild to Moderate Drainage

Used for partial Thickness and shallow full-thickness Wounds

Sterile

How to Apply:

- + Clean and dry skin and wound and surrounding area thoroughly
- + Remove dressing from paper liner, avoid touching the adhesive border
- + Center the dressing over the wound, gently press the adhesive side down
- + Smooth edges down to ensure good adhesion
- + Remove delivery film carefully without lifting edges

Silicone Non-Bordered Foam Dressing

Advanced Wound Care and Healthcare Supplies

The Silicone Non-Bordered Foam Wound Dressing is designed to provide secure yet gentle hold. This soft and conformable foam dressing is indicated for surgical wounds, speeds absorption, and helps to minimize the risk of maceration while the highly absorptive foam wicks exudate away from the wound bed.



REF	UNIT OF MEASURE	CONTENTS	SOLD BY	SIZE
S71044	Box (BX)	10 Each	1 Box of 10	10 × 10cm
S71066	Box (BX)	10 Each	1 Box of 10	15 × 15cm

Product Features:

Silicone Allows for Gentle Removal

Uses as a Primary and Secondary Dressing

Sterile

How to Apply:

- + Clean and dry wound and surrounding skin
- + Select appropriate size dressing to ensure pad is larger than the wound area
- + Remove dressing from package and cut as necessary to accommodate wound
- + Position dressing and gently press to skin
- + **Additional Tip:** Use silicone adhesive and/or tape to secure edges when dressing is used for absorption and cushioning of tube sites.



The Silicone Bordered Foam is a highly absorbent self-adherent silicone foam wound care dressing consisting of a soft silicone contact layer, a flexible polyurethane foam pad, moisture-proof outer film and a vapor-permeable.

Silicone Bordered Foam Dressing

Advanced Wound Care and Healthcare Supplies

REF	UNIT OF MEASURE	CONTENTS	SOLD BY	SIZE
S72044	Box (BX)	10 Each	1 Box of 10	10 × 10cm
S72066	Box (BX)	10 Each	1 Box of 10	15 × 15cm

Product Features:

Silicone Allows for Gentle Removal
Insulates the Wound Bed
Sterile

How to Apply:

- + Clean and dry wound and surrounding skin
- + Select appropriate size dressing to ensure pad is larger than the wound area
- + Remove dressing from package and cut as necessary to accommodate wound
- + Position dressing and gently press to skin
- + **Additional Tip:** Use silicone adhesive and/or tape to secure edges when dressing is used for absorption and cushioning of tube sites.

Calcium Alginate

Alginate wound dressings are non-woven, non-adhesive pads and ribbons composed of natural polysaccharide fibers or xerogel derived from seaweed. On contact with exudate, these dressings form a moist gel through a process of ion exchange. MedSupply Calcium Alginate Dressings are highly absorbent, and made to manage moderate to large amounts of exudate.



Silver Calcium Alginate Dressing

are composed of calcium alginate and silver particles. Silver Calcium Alginate was found to demonstrate an increased antimicrobial effect and increased binding of elastase and cytokines.

REF	UNIT OF MEASURE	CONTENTS	SOLD BY	SIZE
CA4X4	Box (BX)	10 Each	1 Box of 10	10 × 10cm
CA6X6	Box (BX)	10 Each	1 Box of 10	15 × 15cm
SCA4X4	Box (BX)	10 Each	1 Box of 10	10 × 10cm
SCA6X6	Box (BX)	10 Each	1 Box of 10	15 × 15cm

Product Features:

Atraumatic removal

Remains intact when wet

Can be combined with Enzymatic Debridement

How to Apply:

+ Prepare the Skin:

Clean and dry skin to remove lotion, soil, and/or residue.

+ Apply the Tape:

Place dressing on wound

If required place second dressing on top of alginate to hold in place

+ Additional Tips:

Inspect the dressing daily, examining the wound site daily if infected



The Unna Boot with Zinc Oxide & Calamine consists of gauze bandages that have been evenly impregnated with a non-hardening paste of zinc oxide and calamine antipruriginus against irritant, emolient and skin protector to provide venous ulcer compression therapy and help ease itching.

Unna Boot Bandage

Advanced Wound Care and Healthcare Supplies

REF	UNIT OF MEASURE	CONTENTS	SOLD BY	SIZE
Unna Boot	Box (BX)	1 Each	1 Box of 1	10.2cm x 9.1m

Product Features:

Can be wrapped to accommodate several types of limb shapes, all while sitting close to the skin

Formulated with calamine, providing an additional degree of relief in addition to zinc oxide.

Creates a moist healing environment underneath once applied.

Remains in a vacuum-sealed foil pouch until use, ensuring you have a reliable, ready-to-use bandage for longer.

Does not slip

How to Apply:

- + Start Unna Boot Bandage at the base of the toes, keeping the foot and ankle at a right angle.
- + Continue wrapping with a 75% overlap in a spiral fashion while applying light tension. Stop 1-2 inches below the knee.
- + Complete the bandage – smooth and mold with your hands.
- + Cover Unna Boot Bandage with an elastic or cohesive bandage for compression. Maintain an even tension and cover Unna Boot Bandage completely.

**We provide you
with the product,
services, and support
of the highest quality
while maintaining
top-level integrity,
customer satisfaction,
and ethical standards.**



For more information on our products visit our website or
contact us today.

MedSupplyInc.com | 954-342-9720
1241 Stirling Rd. Unit #116
Dania Beach, FL 33004